



Thank you for purchasing the Metro Car Handle. Please read the following instructions for your safety and to learn of the features of this product.

IMPORTANT: Do not attempt to insert the Metro Car Handle into the car door latch in any other direction other than what is shown in the pictures below. Failure to do so may cause injury and could possibly damage your car door latch.

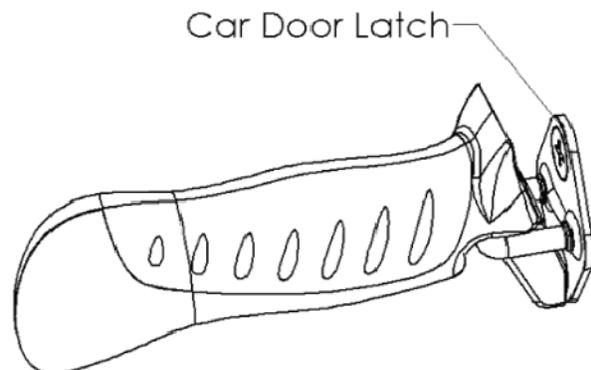


Notice the picture below. This picture represents the product and car door latch in a section view so you can see how the aluminum structure of the Metro Car Handle is inserted to the proper depth in the car door latch before applying any downward force on the handle.

When the Metro Car Handle is inserted into the car door latch as far as possible, the car door latch will engage the recess of the aluminum structure as shown.

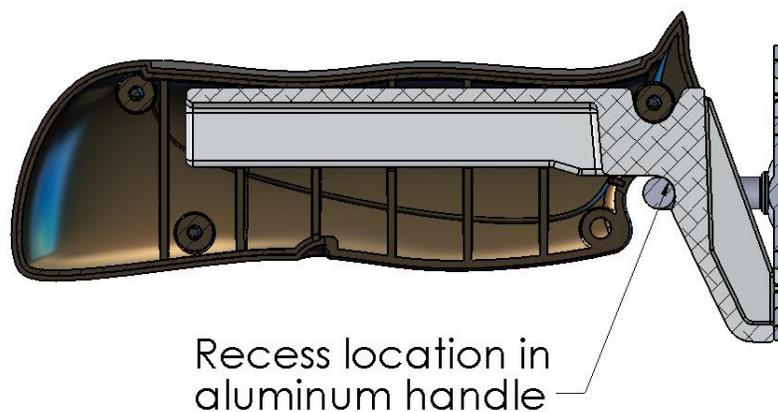

Features:

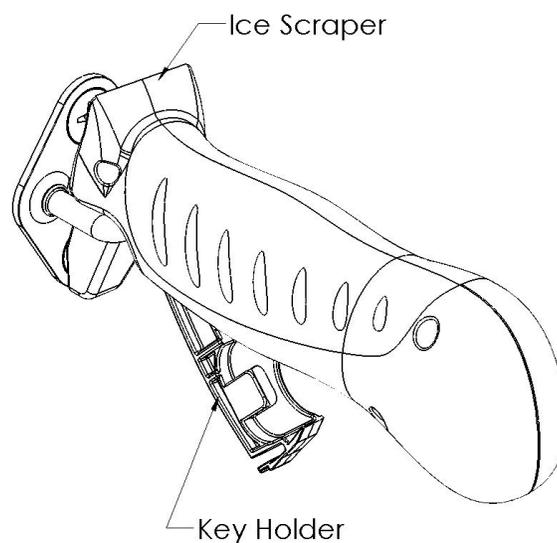

Ice Scraper: Rotate handle 180 degrees so your palm is covering the Key Holder Compartment and use the straight edge as an ice scraper on your car windows.

Key Holder Compartment: The Key Holder Compartment can be opened by pressing the small tab on the Key Holder plastic and rotating the compartment out as shown in the picture above.

WARNING: Please read before using this product.

1. Serious injury or death can result if the Metro Car Handle is not properly installed. DO NOT install the Metro Car Handle in any other direction than described in the instructions. Standers Inc. is committed to providing safe products and informing the user of ways to prevent accidents.

SAFETY PRECAUTIONS:

- ✓ Make sure the Metro Car Handle is properly inserted into the car door latch. The Metro Car Handle must be insert into the car door latch as far as possible before a downward force is exerted.
 - ✓ Additional safety measures should be taken for users that are a high risk. High risk users include those who are not mentally conscious (including those individuals under medication), users who have size or weight conditions, or any condition that impair their ability to use the Metro Car Handle without assistance.
 - ✓ If you have any concerns or questions regarding these precautions, please contact Standers at 1-800-506-9901.
2. If you are suffering from an ailment or injury or are taking medications that affect your balance or ability to stand or sit down, you should first consult with a physician before using this product. If the product is not used properly, serious injury or harm could result.
 3. The product was not intended to carry the full weight of an individual. This product has been tested to support a maximum load of 250 pounds of downward force placed at the midpoint of the handle. In other words a 300 pound person can use this product as long as they do not apply more than 250 pounds of their body weight at the midpoint of the handle. This product is only intended to provide balance and support while sitting and standing.
 4. The product instructions must be carefully followed to ensure that the product is properly assembled and positioned properly. Failure to follow the instructions could result in serious injury to the user(s) of the product.
 5. Children should not be allowed to hang on or play with the product.
 6. **DISCLAIMER:** Neither Standers, Inc. nor the manufacture is liable for any harm or injury that may result or has resulted from the improper use or assembly of the product.

Limited Warranty

Congratulations on your purchase of your Standers brand product. We hope that it will ease your daily living. Please read the following warranty statement for your protection.

Standers, Inc. warrants to the original purchaser that this product will be free from defects in material and/or workmanship for 1 year from the date of purchase.

During the warranty period, Standers, Inc. will correct any defects in material or workmanship, at no charge. The original purchaser, however, must prepay shipping cost. Any replacement parts will be new and are warranted for the remainder of the original warranty or thirty (30) days from the date of shipment, whichever is longer. The original purchaser must notify Standers, Inc. during the warranty period in writing of the defect in workmanship or material. This warranty is not transferable and covers only normal use. **NO SHIPPING CHARGES ARE REFUNDABLE.**

The warranty does NOT cover the following:

- Damage during shipment other than original shipment to retailer.
- Damage caused by unauthorized attachments, alterations, or modifications.
- Damage caused by the use of the product for purposes other than those for which the product was designed or intended.
- Damage caused by any other abuse, misuse, mishandling or misapplication.

Under no circumstances shall Standers, Inc. be liable for any special, incidental, or consequential damages based upon breach of this limited warranty, breach of contract, or strict liability.

THERE ARE NO WARRANTIES WHICH EXTEND BEYOND THE DESCRIPTION ON THEFACE HEREOF. ALL OTHER WARRANTIES, EXPRESS OR IMPLIED (INCLUDING ALL IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE), ARE HEREBY DISCLAIMED.

Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you. This warranty gives you specific legal right, and you may also have other rights, which vary from state to state.

To obtain service under this warranty, you can reach Standers Inc. at 74 Canterbury Circle, Logan, Utah 84321.