


Treating your Neck with a Thermotex Platinum Pad

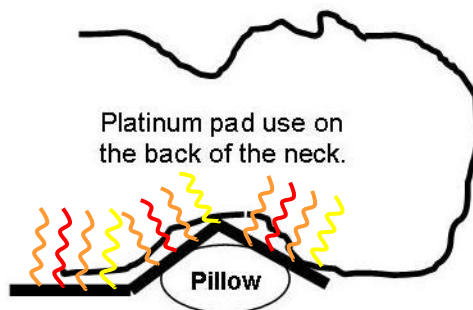

- Use the Thermotex platinum pad with 3 flexible elements.
- Use on the low setting only.
- While on your back place the Thermotex pad as shown
- This method can also be used against a high backed chair
- To treat the side of you neck, simply lay on our side.

MKG11 Rev 0



ActiveForever.com

Discount Therapy Equipment, Medical Equipment & Supplies

SCOTTSDALE 480-767-6800 • **1-800-377-8033** • **PEORIA** 623-215-8470