

Inogen One® G3 Accessories



WARNING Do not use power supplies/adapters or accessories other than those specified in this user manual. The use of non-specified accessories may create a safety hazard and/or impair equipment performance.

Nasal Cannula

A nasal cannula must be used with the Inogen One® G3 to provide oxygen from the concentrator. A single lumen cannula up to 25 feet in length is recommended to ensure proper breath detection and oxygen delivery.



WARNING To avoid danger of choking and strangulation hazard, keep tubing away from children and pets.

NOTE Increasing the cannula length may reduce the perceived noise during oxygen bolus delivery.

NOTE When using a cannula 25ft. in length with the Inogen One® G3, an increase in flow setting may be required.

Carry Bag (CA-300)

The carry bag provides protection while carrying the Inogen One® G3 with a handle and adjustable shoulder strap. The unit is inserted into place from the bottom of the carry bag. Take care to insert the Inogen One® G3 so that the display is seen through the plastic window and the cannula barb is accessible through the open cutout at the top of the bag. Inogen One® G3 intake and exhaust vents should be visible through the open mesh panels. The Inogen One® G3 can be operated using battery power during transport. The expandable zippered bottom is designed with an adjustable zipper for use with a single or double battery and provides easy access for battery changes. The expandable section is not intended for use as a storage compartment.



Inogen One® G3 Optional Accessories

External Battery Charger (BA-303)

1. Plug the External Battery Charger AC power supply cord into an electrical outlet.
2. Plug the External Battery Charger AC power supply into the battery charger.
3. Slide your charger onto the Inogen One G3 Battery by clicking and locking into the charger.
4. When the battery is in the correct position, a solid red light will indicate that the battery is charging.
5. When the green light illuminates, the battery is fully charged.



CAUTION Avoid touching the recessed electrical contacts of the External Battery Charger; damage to contacts may affect charger operation.

NOTE These contacts are not powered unless a battery is in place and charging.

NOTE To completely remove power from the External Battery Charger, remove the plug.

Inogen One G3 Backpack (CA-350)

Alternative/optional way of carrying your Inogen One® G3, hands free, more comfort, out of your way with extra pockets for additional accessories.

